astron® Labtec Reline

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astron® Labtec reline is a resin for relinings, repairs and enlargements in the partial denture- and full denture prosthetics. It is anallergenic, free of MMA and ensures optimal bonding with the base. Universal for use – no special bonding agent required.

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Porcelain mixing pot

When kept in the fridge for a short time (5-10 min.), it provides a longer processing time of the special resin. No release of chemical components!

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1. Place model together with the denture which has to be relined in the relining unit and isolate the denture as usual for counter mold preparation
2. After curing of the stone counter mold dismantle the unit and remove carefully denture from the model
3. Remove impression material from the denture and roughen the denture which has to be relined – e.g. with sandpaper
4. Boil plaster model out and water it
5. Isolate the still warm plaster model
6. Remove the isolation immediately under running tap water and dry it
7. Reposition denture in the counter mold and fix it with e.g. sticky wax
8. Brush Labtec liquid as bonding agent onto the roughened denture
9. Powder and liquid have to be precisely-weighed and then put the appropriate liquid and powder amount for approx. 15 minutes in the refrigerator
10. **Mixing ratio: ideally, use the measuring spoons included in the package**, or 1.0 g powder to 0.85 g liquid or a multiple, using a cold ceramic bowl
11. Spatulate powder slowly and bubble-free into liquid
12. Working time just 2 - 3 minutes
13. Place counter mold with denture in the relining unit
14. Immerse in pressure pot and cure for 60 minutes at a temperature of 52 °C and 2 bar
15. Remove denture mold from pressure pot and prior to open the unit, cool it for 15 minutes under cold running tap water
16. Now open carefully the relining unit, remove denture from plaster model and finish and polish the denture as usual
Attention: Do not steam off the denture, otherwise the high-gloss of the surface will be damaged and becomes matt.
Ultrasonic cleaning and rinsing with fresh water and detergent are sufficient
Hint: for easier removal of the denture from the plaster model, put model for approx. 30 seconds in warm water and remove it carefully

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EC REP KENTZLER-KASCHNER DENTAL GmbH
CH REP RODENT AG
Astron Dental Corp.